en

INSTRUCTIONS FOR USE FOR BAUSCH + LOMB RIGID GAS PERMEABLE CONTACT LENSES

 This leaflet is intended to inform users of Bausch + Lomb Rigid Gas Permeable Contact Lenses how to wear and care for their lenses.

These instructions for use apply to the following products:

| PRODUCT NAME | MATERIAL |
|--|--|
| Boston® ES Boston® EO Boston® XO, Quantum II or ML210 Boston® XO ₂ Boston® Equalens™ Quantum® I or ML92 Optimum Extreme Boston® Equalens™ II Hybrid Fluid Surface or GP Premium | Fluoro-silicone acrylate: Enflufocon A Enflufocon B Hexafocon A Hexafocon B Itafluorofocan A Siflufocon A Roflufocon E Oprifocon A Hybufocon A |
| Boston® II Boston® IV PMMA | Silicone acrylate: Itafocon A Itafocon B Polymethyl methacrylate |

DESCRIPTION AND WEAR APPLICATION

The Rigid Gas Permeable and PMMA Contact Lenses are indicated for daily wear for the correction of refractive ametropia (myopia, hyperopia, astigmatism, and presbyopia) in aphakic and non-aphakic persons with non-diseased eyes.

The lenses may be prescribed in otherwise non-diseased eyes that require a rigid contact lens for the management of irregular corneal conditions such as keratoconus, pellucid marginal degeneration, or following penetrating keratoplasty or refractive (e.g., LASIK) surgery.

The Boston® XO and Boston® XO₂ contact lenses may also be suitable for an orthokeratology fitting program for the temporary reduction of myopia of up to 5.00 diopters in non-diseased eyes.

The Boston® EO contact lens may also be suitable for an orthokeratology fitting program for the temporary reduction of myopia of up to 3.00 diopters in non-diseased eyes.

The Equalens™ contact lens is indicated for daily wear or extended wear from 1 to 7 days between removals for cleaning and disinfection as recommended by the eye care practitioner. It is also indicated for extended wear for the correction of refractive ametropia (myopia and hyperopia) in non-aphakic persons with non-diseased eyes in powers from +12.00D to -20.00D.

The Quantum II RGP contact lens are indicated for daily or extended wear from 1 to 7 days between removals for cleaning and disinfection as recommended by the eye care practitioner, in powers from +20.00D to -20.00D for daily wear and +12.00D to -20.00D for extended wear.

INSERTION AND REMOVAL INSTRUCTIONS

Before handling lenses, always wash and dry hands thoroughly avoiding perfumed or medicated soaps. Carefully remove the lens from the container. Inspect your lens before inserting and do not insert if scratched or damaged.

Insert and remove your lens as instructed by your eye care practitioner.

CARING FOR YOUR LENSES

- Cleaning and disinfecting your lenses between uses is essential for healthy eyes
- Use only Rigid Gas Permeable contact lens care products following the manufacturer's instructions
- Always use fresh, unexpired lens care solutions
- Do not use other solutions such as tap or bottled water or saliva to rinse, store, or re-wet lenses
- Always keep lenses completely immersed in the recommended solution when not being worn
- Never use sharp objects or finger nails to remove lenses from their containers or your eyes
- Remember to clean your lens case whenever you clean your lenses
- Always discard lenses at the end of the wear schedule prescribed by your eye care practitioner
- For extended periods of storage, ensure lenses are cleaned and completely dried and stored in a clean lens case without lens care solution. Thoroughly clean and condition lenses before reusing.

CONTRAINDICATIONS

DO NOT USE RGP and PMMA Contact Lenses if you are suffering from:

- Allergy, inflammation, infection, or redness in or around the eye
- Dry eyes (inadequate tear fluid)
- Poor health affecting the eyes (e.g., colds and flu)
- Any systemic illness affecting the eyes

WARNINGS, PRECAUTIONS AND ADVERSE REACTIONS

Failure to follow your eye care practitioner's recommended wear schedule and cleaning regimen can give rise to greater risk of serious eye infections.

Contact your eye care practitioner before wearing contact lenses when:

- Practicing water sports
- In excessively dry or dusty environments
- Using eye medication.

If you experience any of the following problems, remove your lenses **immediately**:

- Discomfort (eyes stinging, burning, itching, irritation, or other eye pain)
- Unusual secretions or excessive watering from the eye (including allergic reactions)
- Inflammation (redness, conjunctivitis)
- Severe or persistent dry eyes
- Corneal abrasion
- Reduced or blurred vision (visual disturbances or alterations)
- Rainbows or halos around lights
- Sensitivity to light

If any of these symptoms continue after removal, cleaning, and reinsertion of the lens or insertion of a new lens, then remove the lens and **contact your eye care practitioner**. These symptoms can lead to more serious conditions if ignored.

Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision (including blindness).

EMERGENCIES

If chemicals or hazardous substances of any kind (e.g., household products, gardening solutions, laboratory chemicals, etc.) are splashed into your eyes, you should: FLUSH EYES IMMEDIATELY WITH TAP WATER AND THEN REMOVE LENSES PROMPTLY. CONTACT YOUR EYE CARE PRACTITIONER OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.

REPORTING OF SERIOUS INCIDENTS

Any side effects, adverse reactions, or product complaints from use of Bausch + Lomb contact lenses should be reported to the manufacturer. Country specific information can be found on www.bausch.com/contactus

Reporting of serious incidents from use of Bausch + Lomb contact lenses should be reported to the manufacturer and the relevant competent authority.

HOW BAUSCH + LOMB RIGID GAS PERMEABLE CONTACT LENSES ARE SUPPLIED

Each lens is supplied non-sterile in a plastic lens case, dry or in solution (Boston ADVANCE® Conditioning Solution). If supplied dry, clean and condition lenses prior to use.

Bausch + Lomb Incorporated
1400 North Goodman Street
Rochester, NY 14609 USA

Bausch + Lomb Incorporated
Cork Road Industrial Estate
Waterford, X91 V383, Ireland

EU IMPORTER

Bausch + Lomb Netherlands B.V.
Koolhovenlaan 110, 1119 NH Schiphol-Rijk
P.O. Box 75774, 1118 ZX Schiphol
The Netherlands

SYMBOLS & ABBREVIATIONS GLOSSARY

The following may appear on the label of your product:

 **0050** CE Quality certification

DIA Ø₁ Diameter

BC Base curve

PWR F'_V Power

ADD Add power

®/TM are trademarks of Bausch & Lomb Incorporated or its affiliates.

© 2021 Bausch & Lomb Incorporated or its affiliates

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